**(en) HEMATYPE SEGMENT DEVICE**

For Transferring Laboratory Blood Samples From Tubing Segments

**Rx only**

Store at Room Temperature. Protect from freezing. Avoid excessive heat.

Definition of "Room Temperature":

"The temperature prevailing in a working area."

Reference: United States Pharmacopeia, General Notices. United States Pharmacopeial Convention, Inc. 12601 Twinbrook Parkway, Rockville, MD.


**Cautions:**

- Always wear protective safety attire (gloves, lab coat, safety glasses).
- The Hematype Segment Device is for Single Use Only.
- The Hematype Segment Device is for General Laboratory Use Only.
- When inserting segment, always hold Hematype Segment Device and never touch glass test tube.

**Step 1:** Set test tube in rack. Place Hematype Segment Device on top of test tube.

**Step 2:** Cover one end of segment with gloved hand\*, insert other end of segment into Hematype Segment Device and squeeze to obtain sample. (\*Avoids spray from improperly sealed segment.)

**Step 3: Recommended method of segment disposal:** Hold Hematype Segment Device with segment in place and discard together into an approved biohazard medical waste container.

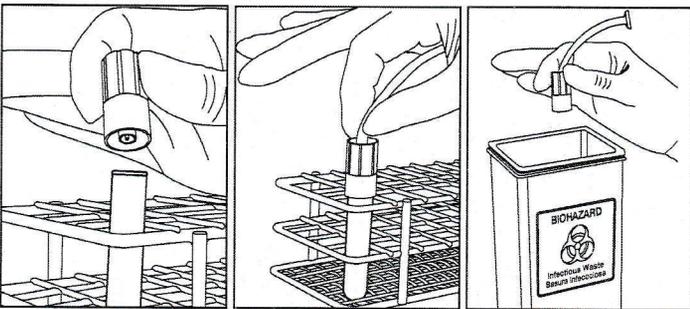Step 1

Step 2

Step 3

**Definition of Symbols:**

- |  |   |  |   |
|--|---|--|---|
|  | Caution, consult instructions for use                           |  | Do not open packaging with sharp objects  |
|  | Do not reuse  |  | Keep dry  |
|  | Do not use if the product sterile barrier system is compromised |  | Stacking limitations  |
|  | Code  |  | Fragile   |
|  | Lot   |  | This way up   |
|  | Manufacturing facility  |  | <b>Rx only</b>  |
|  | Manufacturer  |  | For US only. United States federal law restricts this device to sale by or on the order of a licensed health care practitioner. |
|  | Manufacturing date  |  | © 2021 Fresenius Kabi AG. All rights reserved.  |

**Fresenius Kabi AG**  
61346 Bad Homburg / Germany  
Tel.: +49 (0) 61 72 / 686-0  
www.fresenius-kabi.com  
For US: 1-800-933-6925

**Fenwal International Inc.**  
Carretera Sanchez Km 18.5  
Parque Industrial Itabo  
Haina, Dominican Republic

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© 2021 Fresenius Kabi AG. Tous droits réservés.