


Dimensions	Height	14 inches
	Width	4.50 inches
	Depth	7 inches
Cube		0.25 cu ft
Weight		2.5 lbs
Shipping Dimensions	Height	5.6 inches
	Width	8.25 inches
	Depth	9.45 inches
Cube		0.25 cu ft
Shipping Weight		3.22 lbs
Altitude		0 to 6,562 feet
Operating Temperature Range		64.4° F to 91.4° F (18° C to 33° C)
Storage Temperature Range		-4° F to 149° F (-20° C to 65° C)
Relative Humidity (in use)		20% to 85% Non-condensing
Relative Humidity (storage)		20% to 85% Non-condensing
Power Requirements		12 VAC 50/60 Hz
Amps		500 mA
Watts		6 watts



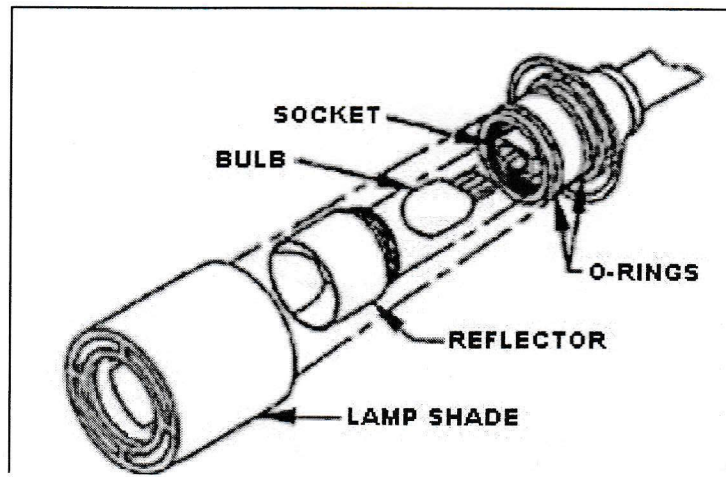

Warning

Do not use this lamp for other than it's intended purpose

- Do not use more than 5watt bulb
- Insert new bulb completely straight into socket
- Touching bulb with out gloves may cause damage to bulb
- Disconnect power to unit before touching bulb
- Do not use near devices that may affected by magnetic field
- Clean with 10% blech with unit disconnected to power
- Clean mirror with non-abrasive cloth
- Wear PPE or eye googles when using unit
- Follow all required specification ranges

Agglutination Viewer


Agglutination viewer is used for observing agglutination reactions in test tubes or other apparatuses used in blood banks. Turn power on to unit and adjust mirror to observe specimen.

Contact Us

Phone: 1-800-948-5024

Email: sales@bloodbankdepot.com

Web: www.bloodbankdepot.com

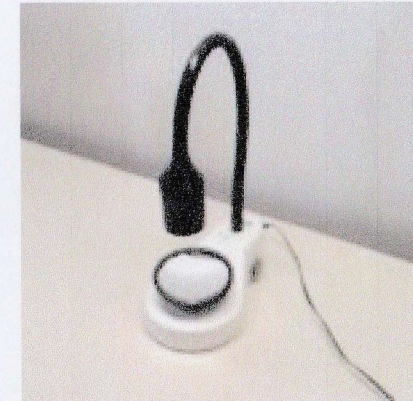

Blood
Depot Inc.

Part# BB22363560



Blood Depot Inc.

4813 Ridge Road STE 111-79
Douglasville, Georgia 30134
United States of America