

Document No.: 472316717
Revision: A
Change Order: CO-046971
Title: PRODUCT INSERT HAND SEALER 4R4454W
Page: 1 of 1
Last Modified: 26 FEB 2024 MKF
Color: Black

PROOFREADING INSPECTION / RELEASED ARTWORK

Proofreading Approval:

Print Name _____

Signature _____

Date _____

Code
REF 4R4454W

1x

Rx only



Fenwal Hand Sealer

The Hand Sealer is used to provide a hermetic closure on Blood-Pack Unit tubings when used with the Fenwal Hand Sealer Clips, code 4R4418.

Sealing Procedure

1. To open the Hand Sealer, squeeze handles until ratchet releases. Handles will open automatically.
2. Fold Blood-Pack Unit tubing as shown in figure below.




Folded Tubing

Cleaning Procedure

Surfaces of the Hand Sealer which have been in contact with blood must be disinfected following established procedures. Remove any residue to ensure that the Hand Sealer will not corrode or discolor.

3. Slide clip over folded tubing and position with end of tubing protruding approximately 1.5 mm. See figure below.
4. Insert tubing with clip attached all the way into crimping jaws of Hand Sealer.
5. Squeeze Hand Sealer until ratchet releases. Handles will open automatically, releasing clip and tubing.
6. Visually inspect the tubing for proper sealing.

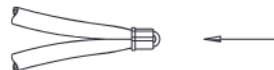

Refer to 3. above

Do not use chemical or abrasive cleaners or sharp edged tools for cleaning.

Do not autoclave the Hand Sealer.

 Manufacturer
Fresenius Kabi AG
Else-Kröner-Str. 1
61352 Bad Homburg
Germany
Tel.: +49 (0) 61 72 / 686-0
www.fresenius-kabi.com

 Manufacturing facility /
Manufactured by
Fresenius Kabi Warrendale
770 Commonwealth Drive
Warrendale, PA 15086
USA
Made in US

 Manufacturer

 Manufacturing facility /
Manufactured by

REF Code

472316717 [A]

©2024 Fresenius Kabi AG. All rights reserved.

